

# SNAP® Feline proBNP Test

## Frequently Asked Questions

### **What is the SNAP® Feline proBNP Test?**

The SNAP® Feline proBNP Test is an in vitro test for the measurement of circulating NTproBNP in feline serum and EDTA plasma.

### **How do I interpret a SNAP Feline proBNP Test result?**

- **Normal Level**

If the color intensity of the sample spot is lighter than the color intensity of the reference spot, NTproBNP levels are normal.

- **Abnormal Level**

If the color intensity of the sample spot is equal to or darker than the color intensity of the reference spot, NTproBNP levels are abnormal.

### **Which types of samples can be used on the SNAP Feline proBNP Test?**

Serum or EDTA plasma, either fresh or stored at 2–8°C for up to one week, can be used.

### **How many sample drops and conjugate drops do I need to run the SNAP Feline proBNP Test?**

The SNAP Feline proBNP Test requires 3 drops of sample and 5 drops of green conjugate.

### **I used the incorrect conjugate/sample ratio. Can I rely on the results I received?**

We recommend rerunning the test with the exact ratio of conjugate/sample listed in the package insert—3 drops of sample and 5 drops of green conjugate.

### **What is the read time for the SNAP Feline proBNP Test and is it really important?**

The test result must be read 10 minutes after the device is activated. The test does not contain stop solution, and after 10 minutes, color development may occur that is not related to the sample. Do not report results read after 10 minutes.

### **How do the kit components need to be stored?**

- Store at 2–8°C until expiration date.
- All components must be at room temperature (18–25°C) before running the test. Do not heat.

### **My SNAP Feline proBNP Test has been out of the foil package for the day.**

#### **Can I still use it?**

The SNAP Feline proBNP Test, and any other SNAP® test, must be used within 2 hours of removing it from the foil package.

### **Can my SNAPshot Dx® Analyzer read the SNAP Feline proBNP Test?**

Yes, the SNAPshot Dx Analyzer can read the SNAP Feline proBNP Test.

### **Can the SNAP Feline proBNP Test be used with SNAP Pro® Mobile Device?**

Yes, the SNAP Feline proBNP Test] can be used with SNAP Pro Mobile Device.