

Using SNAP® Test Kits*

Whole blood/serum/plasma sample test procedure




1. Dispense 3 drops of sample and 4 drops of conjugate (5 drops for cardiac tests) into a disposable sample tube.



2. Gently invert the sample tube 4-5 times to mix.



3. Pour the entire contents of the sample tube into the sample well of a SNAP® device.

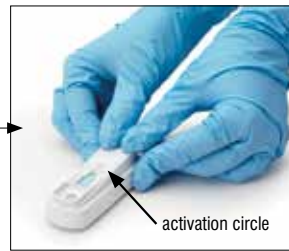

4. When color first appears in the activation circle, press firmly to activate. You will hear a distinct "snap."



5. When the appropriate development time has passed, read the result.

Fecal sample test procedure



1. Swab sample and place swab tip into tube. Bend bulb to break seal and release conjugate.



2. Squeeze and release bulb 3 times to mix sample and conjugate.



3. Squeeze bulb to dispense 5 drops into the sample well of a SNAP® device.




OR Use the SNAP Pro® Mobile Device to automatically activate SNAP tests and display results.

BLUE DOT=POSITIVE
Any color development in the sample spot indicates a positive result.

The SNAP® cPL™ Test, SNAP® fPL™ Test and SNAP® Feline proBNP Test are the exceptions because they provide comparative results.

Interpreting results


Infectious Disease Tests

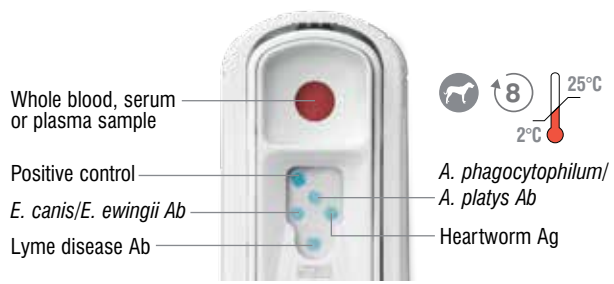

Whole blood, serum or plasma sample
Positive control
E. canis/E. ewingii Ab
Lyme disease Ab

8 25°C
2°C
A. phagocytophilum/
A. platys Ab
Heartworm Ag


SNAP® 4Dx® Plus—Screen dogs annually for six vector-borne diseases.

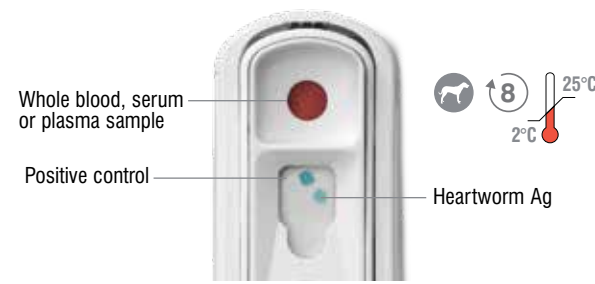

Whole blood, serum or plasma sample
Positive control

8 25°C
2°C
Heartworm Ag


SNAP® Heartworm RT—Screen annually for canine heartworm.

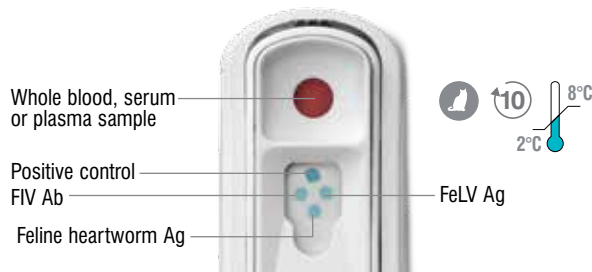

Whole blood, serum or plasma sample
Positive control
FIV Ab
Feline heartworm Ag

10 8°C
2°C
FeLV Ag

SNAP® Feline Triple®—Test all new to household, sick and at-risk cats and cats with respiratory or vomiting symptoms.

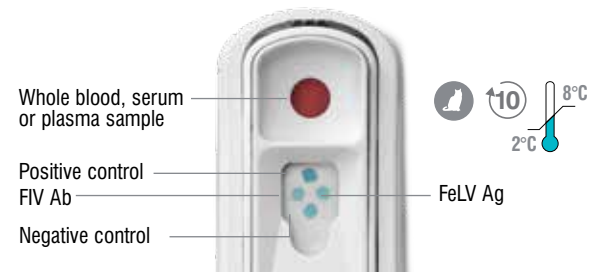

Whole blood, serum or plasma sample
Positive control
FIV Ab
Negative control

10 8°C
2°C
FeLV Ag

SNAP® FIV/FeLV Combo—Test all new to household, at-risk and symptomatic cats.



Fecal sample
Positive control

8 25°C
2°C
Parvovirus Ag

SNAP® Parvo—Test dogs that present with sudden onset of lethargy, vomiting, fever and diarrhea.



Fecal sample
Positive control

8 8°C
2°C
Giardia Ag

SNAP® Giardia—Test dogs and cats that present with diarrhea.



Serum sample
Positive control

10 8°C
2°C
Leptospira Ab

SNAP® Lepto—Test dogs when leptospirosis is suspected.

KEY

- Species
- Development time in minutes.
- Store at room temperature or in refrigerator.
- Store in refrigerator.

ALL components MUST be at room temperature before running the test.

Tech Tip

To help remember how many drops you need, count the number of letters in the color of the liquid:

- R-E-D** 3 drops of blood, serum or plasma
- B-L-U-E** 4 drops of blue conjugate
- G-R-E-E-N** 5 drops of green conjugate
- B-R-O-W-N** 5 drops of fecal mixture

*Before beginning any test, refer to the package insert for complete instructions.

Pancreatic Tests



Serum sample
Reference

10 8°C
2°C
Pancreas-specific lipase

Abnormal
Sample spot is the same color or darker than reference spot.



Serum sample
Reference

10 8°C
2°C
Pancreas-specific lipase

Normal
Sample spot is lighter than reference spot.

SNAP® cPL™—Test dogs that present with vomiting, anorexia and/or abdominal pain.

SNAP® fPL™—Test cats that present with lethargy, decreased appetite, dehydration, weight loss, vomiting and/or diarrhea.

Cardiac Tests



Serum or EDTA plasma sample
Reference

10 8°C
2°C
NTproBNP

Abnormal
Sample spot is the same color or darker than reference spot.



Serum or EDTA plasma sample
Reference

10 8°C
2°C
NTproBNP

Normal
Sample spot is lighter than reference spot.

SNAP® Feline proBNP—Test cats at risk for cardiac disease (murmur, breed, age).



For additional technical information visit idexx.com/snap or call 1-800-248-2483.

To order visit idexx.com/order, call 1-888-79-IDEXX, or talk with your IDEXX Veterinary Diagnostic Consultant.

© 2015 IDEXX Laboratories, Inc. All rights reserved. • 09-60049-17
All ®/TM marks are owned by IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.
The IDEXX Privacy Policy is available at idexx.com.